3M ESPE

Protemp 4

Temporizator Material

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Temporizator

Temporizator

| |
|---|
| Application of a Fittek Flowable Composite to New Temporary Restorations |
| <p>In newly manufactured temporary restorations, the Fittek Flowable Composite is used to reinforce the surface of the unprocessed, but clean, inhibition layer. Bonding is excellent in either case.</p> <ul style="list-style-type: none">Remove any contamination – e.g., from saliva or polishing dust – with alcohol. Then, dry with a stream of air. Cleaning is not necessary if the composite is clean if the temporary restoration is finished and clean. For reasons of hygiene, dose the Fittek Flowable Composite from the Dispenser onto a mixing pad. Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument. Cure each layer separately with light for 20 sec each. Trim the composite with carbide burs and polish, e.g., with Sof-Lex™ polishing disks, manufactured by 3M ESPE. |

| |
|--|
| Application of a Fittek Flowable Composite to Temporary Restorations That Were in Use |
| <ul style="list-style-type: none">Thoroughly roughen the surface of the temporary restoration with rotary instruments. Remove any contamination – e.g., from saliva or polishing dust – with alcohol. Then dry the restoration with a stream of air. For reasons of hygiene, dose the Fittek Flowable Composite from the Dispenser onto a mixing pad. Apply the composite to the temporary restoration in layers of maximally 1 mm using a suitable instrument. Cure each layer separately with light for 20 sec each. Trim the composite with carbide burs and polish, e.g., with Sof-Lex™ polishing disks, manufactured by 3M ESPE. |

| |
|--|
| Lining of Prefabricated, Temporary Crowns Made of Composite and Metal Core build-ups which are fabricated using composites, comonomers, or resin-modified glassionomer cements must be sealed off before the application of Protemp 4, e.g., using Vaseline. |
| <ul style="list-style-type: none">Shape and cure prefabricated, temporary composite crowns (e.g., Protemp™ Crown, manufactured by 3M ESPE) completely. Place comply with the corresponding instructions for Use! Remove any contamination – e.g., from saliva or polishing dust – with alcohol (e.g., ethanol), then dry with a stream of air. Apply Protemp 4 to the prefabricated, temporary crown, see "Application" below for the corresponding instructions for Use! The material attains a hard-elastic consistency within 1 min 40 sec after the onset of mixing must be removed, along with the prefabricated crown, from the mixing within 2 min 50 sec after the onset of mixing. Check the setting of the resin in the mouth. Keep the filled mixing lip on the cartridge to serve as a closure until the next use. Remove the smear layer and finish the residues, see "Finishing". |

| |
|---|
| Cementation |
| <ul style="list-style-type: none">If the restoration has been sealed off (e.g., using Vaseline), remove the rest of the sealant thoroughly before the temporary cementation. Insert the temporary restoration with a commercial temporary cement, e.g., Realyk™ Temp E or ReLyk™ Temp NE, both products manufactured by 3M ESPE – except-containing components may impair the setting of composite luting cements to be used in a later step in the final luting step. If a composite cement is to be used for the final luting step, seal the temporary restoration with a eugenol-free cement. |

| |
|--|
| After Cementation |
| <ul style="list-style-type: none">Thoroughly examine and explore the sulcus and surrounding dentition. Remove any residuals of the temporary material or cement from the mouth. |
| Repair |
| <ul style="list-style-type: none">Provide for mechanical retention by roughening the break surfaces and adjacent areas. Clean with alcohol and dry with air. Can be used for using Protemp 4 or a Fittek Flowable Composite. Cure Protemp 4 or the Fittek Flowable Composite as described in the relevant instructions for Use. Once the material is fully set, finish as usual. |
| Cleaning of the Garant Dispenser |
| <ul style="list-style-type: none">Remove the Garant™ cartridge. Remove paste that has not yet set using an alcohol-soaked cloth. The Dispenser handle and plunger can be autoclaved up to a temperature of 135 °C/275 °F, remove the plunger beforehand. Glutaraldehyde-based solutions can be used for disinfection. |
| Storage and Stability |
| <ul style="list-style-type: none">Store at 2–8 °C/35–47 °F. Do not store in the refrigerator. Do not use after the expiration date. |
| Customer Information |
| <p>No person is authorized to provide any information which deviates from the information provided in this instruction sheet.</p> |
| Warranty |
| <p>3M ESPE warrants this product to be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. This product is not intended for use as a repair or replacement remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.</p> <p> </p> <p>* Not in allen Ländern erhältlich.</p> |

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