coltène®

Lab-Putty hard

Definition

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Lab-Putty hard

Definition

Coltène® Lab-Putty hard is an elastomeric putty for use in the dental laboratory.

Material type

Polysiloxane, condensation-type elastomer.

Indications

Coltène® Lab-Putty hard is exclusively designed for extraoral application.

It has been conceived for the manufacture of:

- Matrix
 - Repro, general occlusion and functional patterns (gfp), matrix as positioner for clasps, denture teeth (i.e. partial dentures, temporary partials, orthodontic retainers etc.), matrix for making gingival masks Coltène® Gi-Mask.
- Repair models (lower dentures with deep undercuts)
- Block-outs
- Hard palate relief stamp (mould)
- Labial/buccal positioner matrix

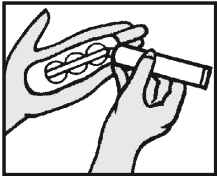
Contra-indications

Provided that the product is used as directed, there are no known contra-indications.

Side effects / Interactions

Polysiloxanes have a very good biological compatibility. Up to now no harmful reactions or secondary effects on laboratory staff are known.

Dosage

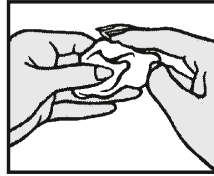
Measure out Coltène® Lab-Putty hard Base with the measuring scoop (take off excess). Spread measured out quantity on the palm of the hand. For each scoop Base press the

scoop-rim once into the mass. Dose one strand-length of Coltène® Lab-Putty Universal Activator per circle-diameter.

Close tube immediately after use.

Mixing

Fold material and knead with fingertips for 45 seconds until a homogeneous, greycolored mixture is produced.

Setting time

Min. 15 minutes at room temperature; min. 7 minutes when used at 37 °C / 98 °F (water bath).

Intensive mixing, high temperatures or over-dosage of the activator speed up the setting.

Low temperatures and under-dosage of the activator slow down the setting.

Max. over-dosage / under-dosage: 10 %

Boilability

The heat resistance when used with heat polymerisation acrylics is guaranteed.

Shelf life and storage

Coltène® Lab-Putty hard fulfills the intended purpose at least until the expiry date, which is displayed on the containers, when stored at 15–23 °C / 59–73 °F and 50 % relative air humidity.

Caution:

Federal law restricts this device to sale by or on the order of a dentist.

Marking

The expiry date and **LOT** number are shown on the package and cartridge.

Technical data

<u>Mixing time:</u>	45 s
<u>Total working time:</u>	120 s
<u>Setting time:</u>	15 min
<u>Shore A hardness:</u>	90
<u>Mixing ration Putty : Activator:</u>	16.0 g : 0.3 g

Mixing conditions:

The stated working time is based on measurements at 23 °C / 73 °F room temperature and 50 % relative humidity.

Date of issue:

11-2011

Only supplied to dentists and dental laboratories or upon their instructions.

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Coltène/Whaledent AG