
Orotol[®] ultra

DE	Sauganlagen-Desinfektion
EN	Disinfection of suction systems
FR	Désinfection des systèmes d'aspiration
IT	Disinfettante per impianti d'aspirazione
ES	Desinfección de dispositivos aspiradores
PT	Desinfecção de sistemas de sucção
NL	Desinfectie van afzuigsystemen
SV	Desinfektionsmedel för suganläggningar
FI	Imulaitteiden desinfiointi
NO	Sugeanleggsdesinfeksjon
DA	Sugeanlægsdesinfektion
CS	Dezinfekce odsávacích zařízení
SK	Dezinfekcia odsávacích zariadení
SL	Dezinfekcija sesalnih naprav
RU	Средство для дезинфекции всасывающих устройств

Gebrauchsanweisung
 Instructions for use
 Mode d'emploi
 Uso
 Instrucciones para el uso
 Instruções de uso
 Gebruiksaanwijzing
 Bruksanvisning
 Käyttöohje
 Bruksanvisning
 Bruksanvisning
 Návod k použití
 Návod na použitie
 Navodilo za uporabo
 Инструкция по эксплуатации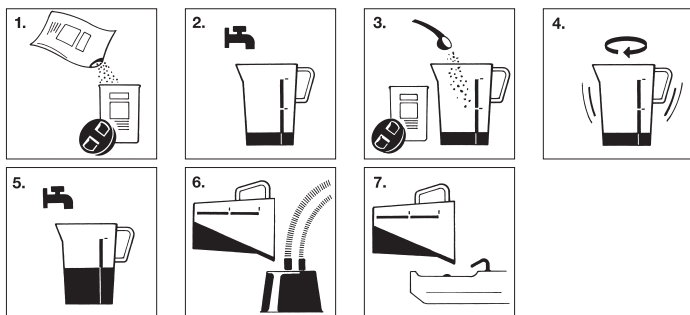

Reizend. Irritant. Irritante. Irriterend. Irritante. Irriterande. Ärsyttävää.
 Irriterende. Lokalirriterende. Dráždí. Dráždívý. Dražijvo. Действует
 раздражающе.

CE 0124

DÜRR DENTAL AG

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Feldstraße 7 a, A-6020 Innsbruck, Austria, ☎ +43 5 12 57 28 34

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Seestraße 108, CH-9326 Horn, ☎ +41 7 18 44 10 30

Orotol® ultra is a highly effective preparation for the simultaneous **disinfection, deodorization, cleaning** and **care** of all dental suction systems as well as spittoon bowls, being likewise suitable for all amalgam separators.

Composition: 100 g Orotol® ultra contain 22 g sodium peroxoborate, alkaline cleansing agents, complexing agents, surfactants, special antifoaming agents, and adjuvants.

Effective spectrum: Bactericidal, tuberculocidal, fungicidal¹⁾, virucidal²⁾ (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV²⁾ as well as non-encapsulated viruses, such as adeno viruses²⁾, polyoma viruses SV 40²⁾, polio viruses²⁾). **List of VAH/DGHH.** Tested in accordance with EN 13727, EN 13624, EN 14561, EN 14562, EN 14563.

Directions for use: Transfer Orotol® ultra from the refill pack into the store bowl (see fig. 1). Orotol® ultra is used as a 1% solution. To 10 g of Orotol® ultra (corresponding to 1 dosing spoon) add lukewarm water (approx. 30° C) to make 1 ltr. and aspirate through suction system using the OroCup or the Orotol® Care System (see fig. 2-6). The exposure time in accordance with VAH/DGHH is 30 minutes. For the inactivation of naked viruses we recommend 1% and 2 hours.

Application	Concentration	Time
Bacteria	1%	30 min.
Tb-bacteria	4%	15 min.
Fungi ¹⁾	1%	15 min.
Enveloped viruses incl. HBV, HCV, HIV ²⁾	1%	5 min.
Naked viruses ²⁾	1%	2 h

¹⁾ Testing with *Candida albicans*.

²⁾ In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

We recommend disinfecting and cleaning the suction system with Orotol® ultra once or twice a day, before lunch-time and after surgery ends for the day, depending on work-load. Once-daily disinfecting and cleaning should be done at the end of the day exclusively. If necessary, disinfect and clean spittoon bowl with Orotol® ultra, too (see fig. 7). Prepare a fresh ready-to-use solution for each application.

Ecology: In compliance with the OECD guidelines 301 D, Orotol® ultra is biodegradable.

Caution: After surgical procedures, aspirate a glass of cold water intermittently. No foaming agents must enter the suction system or the spittoon bowl.

Shelf life: Concentrate 2 years, ready-to-use solution 1 day.

Ch.-B./ expiry date: see imprint on refill pack

Irritating to the skin. Risk of serious damage to eyes. In the event of contact with the eyes, rinse them thoroughly with water and consult a doctor. We recommend the wearing of protective gloves.

Packing units:

Order No.: CDS120U99- **Basic set** for first use including 1 store bowl, 2 x 500 g Orotol® ultra, 1 dosing spoon, 1 OroCup.

Order No.: CDS120U67- **Refill pack** containing 8 x 500 g Orotol® ultra, 1 dosing spoon.

Distributor: DÜRR DENTAL (Products) UK Ltd.

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