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EN ID 212 Instrument disinfection

ID 212 is an aldehyde-free concentrate for the disinfection and cleaning of general dental instruments (mirrors, probes, tweezers, forceps etc.). An additional field of application is the disinfection and cleaning of alkali- and alcohol-sensitive rotary instruments.

Composition: 100 g ID 212 contain 9 g alkyl-benzyl-dimethyl-ammoniumchloride, 0,1 g guanidine compounds, non-ionic surfactants, alkaline detergents, complexing agents, benzyl salicylate and accessory agents.

Effective spectrum: Bactericidal, tuberculocidal, fungicida¹⁾, limited virucidal activity²⁾ (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV as well as non-encapsulated viruses, such as noro viruses³⁾. **Liste VAH/DGHH.** Tested in accordance with EN 13727, EN 13624, EN 14561, EN 14562.

1) Testing with Candida albicans.

2) In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

3) Test for MNV surrogate virus.

Directions for use: ID 212 is used as a 2% solution. To 20 ml of ID 212 add 980 ml of water. The exposure time is 5 minutes, for TB bacteria 60 minutes. For use in **ultrasonic equipment**, we recommend 2% for 2 minutes, for TB bacteria 2% and 60 minutes. Do not mix ID 212 with ordinary household detergents. Make up the solution for use with lukewarm water. Rinse disinfected instruments in running water, dry and sterilize, if necessary. The maximum standing time for the ready-for-use solution is 7 days. When the contamination with blood, saliva etc. is high we recommend renewal of the solution twice or three times per week.

Ch.-B./expiry date: see bottle imprint.

Irritant. Dangerous for the environment. Irritating to skin. Risk of serious damage to eyes. Very toxic to aquatic organisms. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable gloves and eye/face protection. Use appropriate container to avoid environmental contamination. Avoid release to the environment. Refer to special instructions/Safety data sheets.

DÜRR DENTAL (Products) UK Ltd. 14 Linnwell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, ☎ +44 1536 526740

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Forhandels i Norge av: A/S NORSK DENTAL DEPOT Østønsjøveien 40, N-0667 Oslo 6, ☎ +47 22072727