# MD 535

D	Gipsentferner
GB	Plaster remover
F	Dissolvant de plâtre
I	Solvente per gesso
E	Eliminador de yeso
P	Removedor de gesso
NL	Gipsverwijderaar
S	Gipsborttagare
FIN	Kipsinpoistoaine
N	Gipsfjerner
DK	Gipsfjerner
GR	Kaθαριστικό γύψου
PL	Środek do usuwania gipsu
CZ	Prostředek na odstraňování sádry
SK	Odstraňovač sadry
SLO	Odstranjevalec mavca
H	Gipszeltávolító
EST	Kipsieemaldaja
LV	Ģipša tīrišanas līdzeklis
LT	Priemonė gipsui šalinti
RUS	Средство для удаления гипса
BG	Препарат за отстраняване на гипс
RO	Produs pentru îndepărtarea ghipsului

DÜRR DENTAL AG  
74302 Bietigheim-Bissingen,  
Germany  
Tel: +49 7142 705-0  
info@duerr.de • www.duerr.de  
Made in Germany

## MD 535 Gipsentferner

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## MD 535 Plaster remover

MD 535 is a special cleaner with strong cleaning power for the removal of **plaster** and **alginates** from impression trays, spatulas, plaster knives, mixing beaters (also from vacuum mixing devices), prosthetic materials of all types, orthodontic appliances, etc.

**Composition:** MD 535 contains complexing agents (15 - 30%), alkaline cleansing agents (< 5%) and auxiliary substances.

**Instructions for use:** MD 535 is used undiluted. Place the parts to be cleaned in MD 535 and allow the solution to take effect. After treatment, rinse off the cleaning agent with water.

The cleaning process is accelerated by the use of an **ultrasonic unit** and by heating to up to a maximum of 50 °C. Renew solution when necessary. Rising gas bubbles indicate a cleaning action. If these are no longer present, a fresh solution must be used. Residues in the MD 535 solution do not influence its action.

**Ch.-B./expiry date:** see imprint.

**Irritant:** Risk of serious damage to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Wear suitable gloves and eye/face protection.

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