Enamelast® Fluoride Varnish

5% Sodium Fluoride – Syringe Delivery

Description:
Enamelast® Fluoride Varnish is a flavored, xylitol-sweetened 5% sodium fluoride in a resin carrier delivered in a 1.2mL syringe.

Indications for Use:
Enamelast® Fluoride Varnish is a 5% sodium fluoride in a varnish carrier which produces a mechanical occlusion of the dentinal tubules in the treatment of tooth hypersensitivity.

Directions for Use:

1. Thoroughly brush teeth, rubber cup polish or wipe teeth with gauze prior to placement.
2. When applying directly from the 1.2mL syringe, remove the cap and securely attach the SoftEZ delivery tip (sold separately).
3. Verify flow prior to applying intraorally.

NOTE: Enamelast can also be placed on the back of the clinician's gloved hand or in a dappen dish and applied with a standard brush (sold separately).

4. Lightly dry area to be treated.
5. Apply a thin smooth layer to as many dry tooth surfaces as possible using a painting motion.

NOTE:

- Enamelast sets when it contacts with water or saliva. Allow cheeks, lips and saliva to contact teeth or gently flow cool water over the teeth. For esthetic reasons, encourage patients not to "lick" teeth for a few minutes after placement.
- Patients should avoid brushing, flossing and chewing hard foods and hot foods for 4-6 hours after treatment.

Enamelast® Fluoride Varnish will begin to flake off with each brushing.

Precautions and Warnings:

1. Carefully read and understand all instructions before using.
2. Do not use more than 1/2 of the 1.2mL syringe of fluoride varnish per patient per application.
3. Store at room temperature.
4. Verify flow prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.
5. Do not remove cap until ready to use.
6. Patient, clinician and assistant must wear protective eyewear when using this product.
7. Do not use on patients with ulcerative gingivitis or stomatitis.
8. Do not use with other topical fluoride treatments the same day and instruct patient to refrain from using oral fluoride supplements for 4 (four) days after treatment.
9. Do not use on patients with a known sensitivity to ingredients including colophony (kolophonium) and/rosin. If allergic reaction, dermatitis or rash develops, consult a physician.
10. In rare instances, edematous swelling has been reported after extensive fluoride varnish treatment and dyspnea has been reported in asthmatic children.
11. Some patients may experience nausea after application. If nausea persists, thoroughly brush to remove the varnish film.
12. Syringe tips are for single use only, to avoid cross-contamination.
13. The pre-filled syringes can be used repeatedly. During use, they have to be protected by Syringe Covers. After use, remove and dispose the Syringe Cover, together with the syringe tip. Re-cap syringe with the Luer Lock Cap and disinfect the syringe with a disinfecting agent, suitable for medical devices, which has no protein-fixing effect.
14. Dispose of tips and empty syringes properly.
15. Keep out of reach of children.
16. For professional use only.

NOTE: For MSDS and additional information about using Enamelast® Fluoride Varnish and other related products, please go to www.ultradent.com

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