Enamelast® Fluoride Varnish
5% Sodium Fluoride Unit Dose

Description: Enamelast® Fluoride Varnish is a flavored, xylitol-sweetened 5% sodium fluoride in a resin carrier delivered in a 0.4mL unit dose blister pack delivery.

Indications for Use:

Enamelast® Fluoride Varnish contains 5% sodium fluoride in a varnish carrier which produces a mechanical occlusion of the dentinal tubules in the treatment of tooth hypersensitivity.

Directions for Use:

- Thoroughly brush teeth, rubber cup polish or wipe teeth with gauze prior to placement.
- Open blister pack by peeling back and removing foil lid.
- Lightly dry area to be treated.
- Using the disposable delivery brush, apply a thin smooth layer of Enamelast to as many dry tooth surfaces as possible using a painting motion.

NOTE:

- The brush may be bent up to a 90 degree angle for easier access.
- Enamelast sets when in contact with water or saliva. Allow cheeks, lips and saliva to contact teeth or gently flow cool water over the teeth. For esthetic reasons, encourage patients to NOT "lick" teeth for a few minutes after placement.
- Patients should avoid brushing, flossing and chewing hard, sticky and hot foods for 4-6 hours after treatment. Enamelast® Fluoride Varnish will begin to flake off with each brushing.

Precautions and Warnings:

- Carefully read and understand all instructions before using.
- Store at room temperature.
- Do not use on patients with ulcerative gingivitis or stomatitis.
- Do not use with other topical fluoride treatments the same day and instruct patient to refrain from using oral fluoride supplements for 4 (four) days after treatment.
- Do not use on patients with a known sensitivity to ingredients including colophony (kolophonium) and/rosin. If allergic reaction, dermatitis or rash develops, consult a physician.
- In rare instances, edematous swelling has been reported after extensive fluoride varnish treatment and dyspnea has been reported in asthmatic children.
- Some patients may experience nausea after application. If nausea persists, thoroughly brush to remove the varnish film.
- Properly disposed of containers and brushes after use.
- Keep out of reach of children.
- For professional use only.

NOTE: For MSDS and additional information about using Enamelast® Fluoride Varnish and other related products, please go to **www.ultradent.com**

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