

zeta 7 solution

IMPRESSIONS



LIQUID DISINFECTANT FOR IMPRESSIONS

CONCENTRATE 1%

Bactericidal: EN 13727 (S. Aureus, P. Aeruginosa, E. Hirae)
Fungicidal: EN 13624 (C. Albicans)
Tuberculocidal: EN 14348, EN14563 (M. Terrae)
Virucidal, including HIV, HBV, HCV:
EN 14476 (Poliovirus, Adenovirus, Parvovirus)



Reorder N.
REF C810049

5000 ml e



Zhermack®
CLINICAL



Via Bovazecchino, 100 - 45021 Badia Polesine (Ro) - Italy
Tel. +39 0425 597 611 - www.zhermack.com

Made in Italy

EN - Concentrate disinfectant liquid for impressions

Zeta 7 Solution is an aldehyde-free concentrated disinfectant with a wide range of action prepared and tested in compliance with new harmonised European disinfection regulations. Zeta 7 Solution has been designed for high-level, rapid disinfection of impressions (addition and condensation), in silicone, alginate, polyether and polysulphur: the impressions remain unaltered, have no significant dimensional change and are compatible with plasters.

Composition:

100 g of Zeta 7 Solution contains 7.7 g dimethyl-didecyl-ammonium chloride, 15 g phenoxyethanol, non-ionic surfactants, additives, auxiliaries and water to 100 g.

Range of action:

Bactericidal, fungicidal (test on *Candida Albicans*), tuberculocidal and virucidal. Suspension activity tested in accordance with EN13727, EN13624, EN14348. Surface activity tested in accordance with EN14563. Virucidal activity tested in accordance with EN14476.

Use:

Immediately after taking the impression, rinse it under running water for 30 seconds. Place the impression for 10 minutes in a 1% solution bath of Zeta 7 (use the dispenser cap to measure 10 ml of Zeta 7 Solution to be added to every litre of water). Remove the impression from the bath and rinse it thoroughly. Follow recommended immersion times as suggested. Replace the solution frequently, especially where there is a high level of organic contamination (blood, saliva).

| | | |
|----------------------|--------------|--|
| Suspension activity: | 1 minute | bactericidal (EN13727) |
| | 1 minute | fungicidal (EN13624, test on <i>Candida Albicans</i>) |
| | 1 minute | tuberculocidal (EN14348) |
| Surface activity: | 2.30 minutes | tuberculocidal (EN14563) |
| Virus activity: | 10 minutes | "dirty" conditions (EN 14476) |

Warnings, precautions and conservation conditions:

This product has been designed for use by qualified operators with knowledge of the required safety standards. Use protective gloves. Operate in a well-ventilated area. Store the closed container in a cool dry place. Prevent Zeta 7 Solution from coming into contact with other disinfectants or cleaners.

Safety instructions:

R34: Causes burns. - R22: Harmful if swallowed. - R41: Risk of serious damage to eyes. - R36/38: Irritating to eyes and skin.