


# Variolink® Esthetic DC


## Placement of inlays (glass ceramic, composite)

### Try-in


Variolink Esthetic Try-in

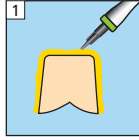


Application of Try-in paste

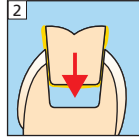


Try-in of restoration

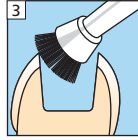


Cleaning after Try-in

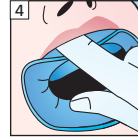


Isolation

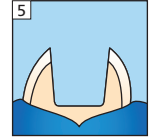

### Restoration

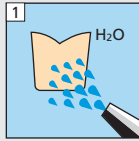

H<sub>2</sub>O

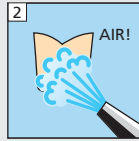

AIR!

#### Pre-treatment of ceramic restorations:

According to manufacturer's instructions

E.g. IPS Empress: Etch for 60 s


IPS e.max CAD/Press: Etch for 20 s with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.

#### Pre-treatment of composite restorations:

According to manufacturer's instructions

E.g. SR Nexco, 2 bar, Al<sub>2</sub>O<sub>3</sub> 100 µm or roughened with finishing diamonds.

Monobond Plus

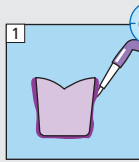


60 s




AIR!

Total Etch optional


### Preparation

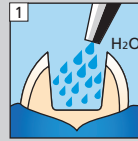

H<sub>2</sub>O

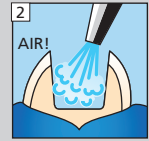


AIR!

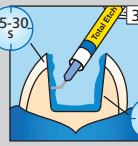


15-30 s

3

15 s

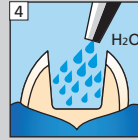


H<sub>2</sub>O

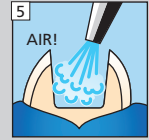

AIR!



Adhese Universal

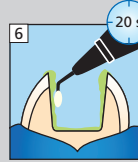

20 s




AIR!

Avoid pooling





Bluephase Style

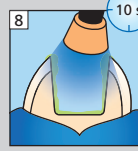


10 s

### Cementation

Variolink Esthetic DC

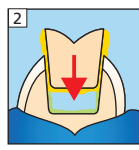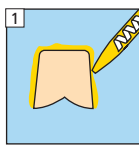


Bluephase Style

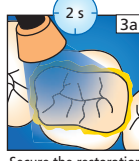


2 s

3a

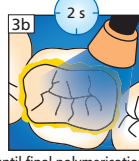


2 s

3b

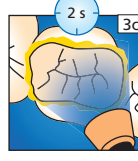


2 s

3c

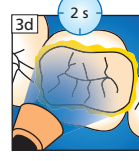



2 s

3d

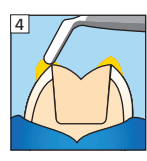


Secure the restoration until final polymerisation

Liquid Strip

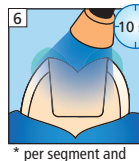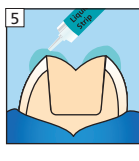


\* per segment and mm ceramic

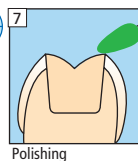

Polishing

Fluor Protector

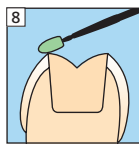

more information  
www.cementation-navigation.com

CE 0123



Please read  
Instruction for Use

ivoclar  
vivadent  
clinical

Date information prepared: 2014-06-25  
666357/e/Rev. 0

# Variolink® Esthetic DC


Placement of crowns and bridges  
(glass ceramic, oxide ceramic, composite)

## Try-in

Variolink Esthetic Try-in



## Restoration

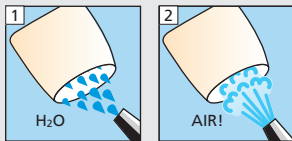


**Pre-treatment of glass ceramic restorations:**  
According to manufacturer's instructions  
E.g. IPS Empress: Etch for 60 s  
IPS e.max CAD/Press: Etch for 20 s  
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.



**Pre-treatment of oxide ceramic restorations:**  
According to manufacturer's instructions  
E.g. IPS e.max ZirCAD  
max. 1 bar, Al<sub>2</sub>O<sub>3</sub> 100 µm

**Pre-treatment of composite restorations:**  
According to manufacturer's instructions  
E.g. SR Nexco, 2 bar, Al<sub>2</sub>O<sub>3</sub> 100 µm  
or roughened with finishing diamonds.


Monobond Plus

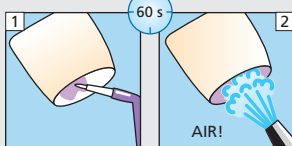

Total Etch  
optional

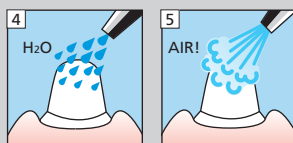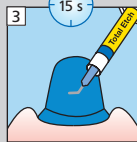


Adhese Universal

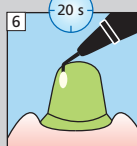


Avoid pooling

Bluephase Style

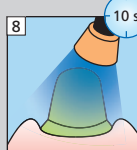


## Cementation

Variolink Esthetic DC

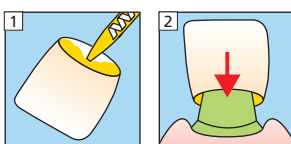


Bluephase Style

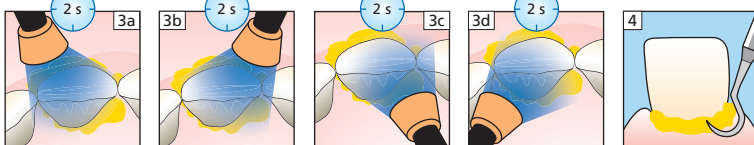


Secure the restoration until final polymerisation

Liquid Strip

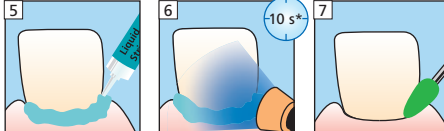

\* per segment and mm ceramic

Fluor Protector

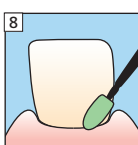

more information  
[www.cementation-navigation.com](http://www.cementation-navigation.com)

CE 0123



Please read  
Instruction for Use

ivoclar  
vivadent  
clinical