


Variolink® Esthetic LC


Placement of Veneers (glass ceramic, composite)

Try-in


Variolink Esthetic Try-in

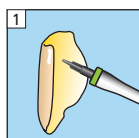


Application of Try-in paste

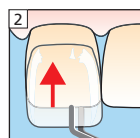


Try-in of restoration

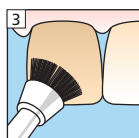



Cleaning after Try-in

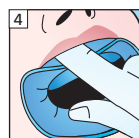

Isolation

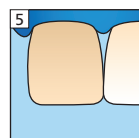



Restoration

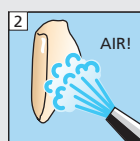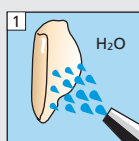


Pre-treatment of ceramic restorations:
According to manufacturer's instructions
E.g. IPS Empress: Etch for 60 s
IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.

Pre-treatment of composite restorations:
According to manufacturer's instructions
E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.



Monobond Plus

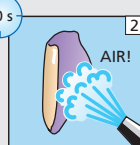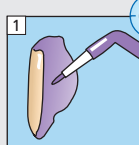



Preparation

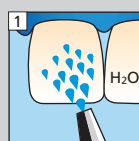

Total Etch
optional

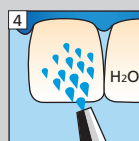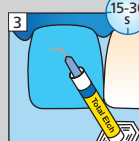

Adhese Universal

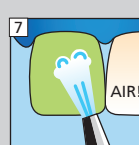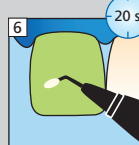



Avoid pooling

Bluephase Style




Cementation

Variolink Esthetic LC

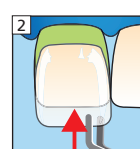
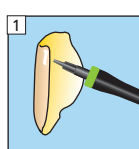


Bluephase Style

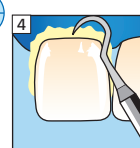


Secure the restoration until final polymerisation


Liquid Strip

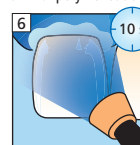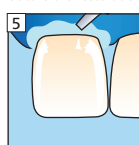

Polishing

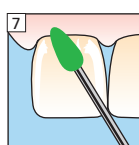

Fluor Protector

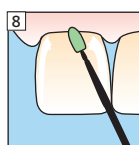

Date information prepared: 2014-06-25
666332/e/Rev. 0



more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use


ivoclar
vivadent
clinical

Variolink® Esthetic LC


Placement of inlays (thickness < 2 mm) (glass ceramic, composite)

Try-in


Variolink Esthetic Try-in

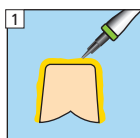


Application of Try-in paste

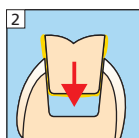


Try-in of restoration

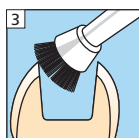


Cleaning after Try-in

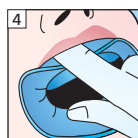

Isolation

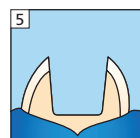


Restoration

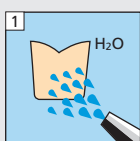


Pre-treatment of ceramic restorations:
According to manufacturer's instructions
E.g. IPS Empress: Etch for 60 s
IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.

Pre-treatment of composite restorations:
According to manufacturer's instructions
E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.



Monobond Plus

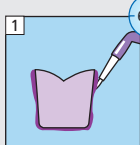


Preparation

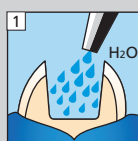

Total Etch
optional

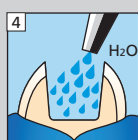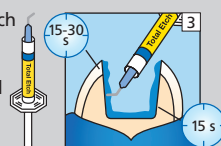


Adhese Universal

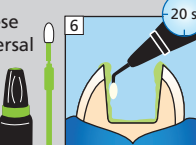



Avoid pooling

Bluephase Style


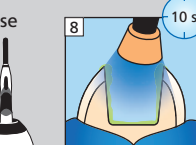


Cementation

Variolink Esthetic LC


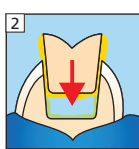
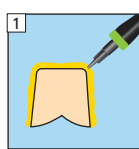


Bluephase Style

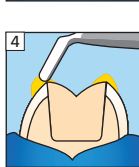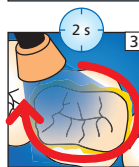


Secure the restoration until final polymerisation

Liquid Strip

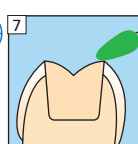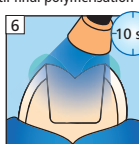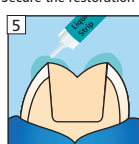

* per segment and mm ceramic

Fluor Protector

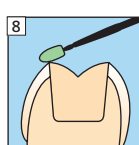

Polishing



more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical