F

MM-EDTA CREAM

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GB

MM-EDTA CREAM

EDTA Cream for cleaning and shaping root canals.
Lubricates for smoother endodontic file penetration

INDICATIONS

- Preparing root canal using endodontic files
- Shaping calcified root canals
- Preparing narrow canals

COMPOSITION: EDTA: 19%, Polyethylene glycol: 81%

MODE OF ACTION

- Removes inorganic substances by chelation (EDTA-Ca²⁺)
- Enhances file penetration due to its lubricating action
- Makes it easier to shape calcified canals

COMPONENTS

- MM-EDTA Cream 5.4ml x 2
- Disposable, single-use tips
- Dimensions: 195 x 70 x 30 mm
- Net weight: 14 g

DIRECTIONS FOR USE

Attach a tip to the syringe. Deposit a small amount of MM-EDTA Cream in the canal orifice using the syringe. Carefully push the MM-EDTA Cream into the canal using a fine file.

Re-apply MM-EDTA Cream as necessary or until cleaning and shap-ing is complete. Remove the MM-EDTA Cream by thorough irrigation with sodium hypochlorite. Remove the used tip, attach a new one, and cap the syringe.

PRECAUTIONS

- Use just enough MM-EDTA Cream to lubricate the files and pre-vent ledging or perforation.
- Avoid extrusion of the cream into the apex.
- Keep the syringe capped all the time when not in use.
- Store at 10°C–25°C (50°F–77°F).

CLEANING AND PROTECTION

Disposable tip should be thrown away.
The syringe must be protected against cross-contamination using a disposable barrier sleeve.

WARNINGS

The syringe must be protected against cross-contamination using a disposable barrier sleeve.

EXPIRY DATE

Two (2) years after the manufacturing date. Do not use after the expiry date.

SYMBOLS

: Use by (64°F-75°F)

: Batch code

: Attention, see instructions for use

: Store at 10°C – 25°C

Caution: US Federal law restricts this device to the sale by or on the order of a dentist.

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