SYMBOLS DESCRIPTION

|  |                                                                        |
|--|------------------------------------------------------------------------|
|  | Warning or Caution                                                     |
|  | Consult Instructions for Use                                           |
|  | Temperature Storage Limitations:<br>Low 2° C/35° F and High 8° C/46° F |
|  | Manufacturer                                                           |
|  | Use-by Date                                                            |
|  | Do not re-use                                                          |
|  | Date of manufacture                                                    |
|  | Manufacturing lot number                                               |
|  | European Authorized Representative                                     |
|  | Part Number                                                            |

en - ENGLISH

**OptiBond™ Universal**  
Single-Component Universal Adhesive

DEVICE DESCRIPTION

OptiBond™ Universal is formulated to provide reliable bond strengths in any etch technique, including self etch, selective enamel etch, and total etch. This flexibility allows the clinician to choose a technique based on either the indication or personal preference, all with a single adhesive system.

INDICATIONS FOR USE

Direct Applications

- Light-cured composite and compomer restorations.
- Composite/ceramic/metal repairs.
- Cavity sealing for amalgam restorations.
- Light-cured or dual-cure core build-ups (always light-cure the first incremental build-up of core material).

Indirect Applications

- Veneers.
- Porcelain, composite, and metal-based inlays, onlays, crowns and bridges.
- Endodontic posts.
- Cavity sealing as a pretreatment for indirect restorations.
- OptiBond Universal adhesive can be used to prime metal, ceramic, and composite substrates/restorations. This eliminates the need for other primers such as silane or metal primer.

**IMPORTANT:** For indirect applications, use NX3 resin cement whenever there is or no limited light accessibility. NX3, either self-cured or dual-cured, is compatible with OptiBond Universal adhesive.

CONTRAINDICATIONS

- Patients who have a history of severe allergic reaction to methacrylate resins.
- Direct pulp capping.
- In combination with a chemical-cure or dual-cure resin material (other than NX3) when there is insufficient light accessibility.

WARNINGS

Do not use on patients with known methacrylate allergies. Uncured methacrylate resin may cause contact dermatitis and damage to the pulp. OptiBond Universal adhesive contains acetone and is flammable. Keep away from sources of ignition. OptiBond Universal is a corrosive liquid. Gel Etchant is a corrosive acid - contains phosphoric acid. Avoid contact with skin, eyes and soft tissue. In case of contact with skin or eyes, flush immediately and thoroughly with water. Get medical attention for eyes. Do not take internally.

PRECAUTIONS

Refrigerate upon receipt between 2° and 8°C. OptiBond Universal adhesive bottle should be tightly closed immediately after use. OptiBond Universal adhesive is a light-cured material. Once the material is placed in a mixing well, use immediately or protect from ambient light.

**Gel Etchant syringe:** Never use excessive force on syringe plunger to dispense etchant which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extrudarily.

ADVERSE REACTIONS

None Known.

INSTRUCTIONS FOR USE

Step-by-Step Instructions

Direct Restorations

Bonding Composites/Compomers to Dentin/Enamel

- Prepare cavity. Wash thoroughly with water spray and air dry (do not desiccate).

For Self-Etch Technique, omit Steps 2-3 and proceed to Step 4.

For Selective Enamel Etch Technique

- A - Etch enamel margins with a phosphoric acid gel etchant (e.g. Kerr's 37.5% Gel Etchant) for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed to Step 3.

For Total Etch Technique

- B - Etch enamel and dentin cavosurfaces with a phosphoric acid gel etchant (e.g. Kerr's 37.5% Gel Etchant) for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed to Step 3.
- Air dry lightly; do not desiccate.
- Shake OptiBond Universal adhesive bottle briefly.

**Note:** Upon first use, shake OptiBond Universal adhesive bottle vigorously for 10 seconds.

- Bottle Delivery:** Dispense 2-3 drops of OptiBond Universal adhesive into a clean well. Replace cap immediately after dispensing.

Unidose™ Delivery

**For single patient use only:** Open the unidose device and insert the applicator brush into the container to saturate the tip.  
6. Using the disposable applicator brush, apply a generous amount of OptiBond Universal adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds.

- Dry the adhesive with gentle air first and then medium air for at least 5 seconds with oil-free air. The surface should have a glossy uniform appearance. If not, repeat steps 6 and 7.
- Light cure for 10 seconds.\*
- In combination avec le matériau à base de résine à polymérisation chimique ou double polymérisation (autre que NX3) en cas de photopolymérisation insuffisante.

Indirect Restorations

(See indication-specific information in Section A-D below):

Restoration Surface Preparation:

**Ceramic/Porcelain Restorations:** Sandblast the surface with 50µ alumina with a pressure of about 30 psi (0.2 MPa) and etch the restoration with hydrofluoric acid for 1 minute. Apply a coat of OptiBond Universal adhesive† on the internal surface of the restoration, air thin with gentle air first and then strong air to avoid pooling of OptiBond Universal adhesive, and light-cure for 10 seconds\* (light-curing of OptiBond Universal adhesive is optional if NX3 resin cement is used for cementation).  
†Alternatively, apply a coat of Kerr Silane on the internal surface of the restoration, and lightly air thin. Protect the restorations from light as Kerr Silane contains resin and is light-sensitive.

**Metal-based, Zirconia-based, Alumina-based and Composite Restorations:** Sandblast the internal surface with 50µ alumina with a pressure of about 60 psi (0.4 MPa) for metal-based, zirconia-based and alumina-based restorations, or a pressure of about 15 psi (0.1 MPa) for composite restorations. Apply a coat of OptiBond Universal adhesive on the internal surface of the restoration, air thin with gentle air first and then strong air to avoid pooling of OptiBond Universal adhesive, and light-cure for 10 seconds\* (light-curing of OptiBond Universal adhesive is optional if NX3 resin cement is used for cementation).

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- A - Etch enamel margins with a phosphoric acid gel etchant (e.g. Kerr's 37.5%

Gel Etchant) for 15 seconds, then rinse thoroughly en suring that all etchant has been removed. Proceed to Step 3.

For Total Etch Technique

- B - Etch enamel and dentin cavosurfaces with a phosphoric acid gel etchant (e.g. Kerr's 37.5% Gel Etchant) for 15 seconds, then rinse thoroughly ensuring that all etchant has been removed. Proceed to Step 3.
- Air dry lightly; do not desiccate.
- Shake OptiBond Universal adhesive bottle briefly.

**Note:** Upon first use, shake OptiBond Universal adhesive bottle vigorously for 10 seconds.

- Bottle Delivery:** Dispense 2-3 drops of OptiBond Universal adhesive into a clean well. Replace cap immediately after dispensing.

Unidose™ Delivery

**For single patient use only:** Open the unidose device and insert the applicator brush into the container to saturate the tip.  
6. Using the disposable applicator brush, apply a generous amount of OptiBond Universal adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds.

- Dry the adhesive with gentle air first and then medium air for at least 5 seconds with oil-free air. The surface should have a glossy uniform appearance. If not, repeat steps 6 and 7.
- Light cure for 10 seconds.\*

III. Sealing/Insertion of Restoration

- Veneers** - Apply luting cement to veneer, seal the veneer, and cure the cement according to manufacturer's instructions for use.
- Crowns, bridges, inlays, onlays and metal-based restorations** - Avoid pooling of adhesive before light curing. Following light curing of the adhesive, apply luting cement (see important note below for cement recommendation) to the restoration or prep, seal the restoration, and cure the cement according to manufacturer's instructions for use.

**Important Note:** Kerr recommends the use of NX3, Maxcem Elite™, or Maxcem Elite Chroma resin cement whenever there is or no limited light accessibility (e.g. metal based, zirconia-based and alumina-based substrates).

- Post Bonding** - Apply adhesive on post and prep. Avoid pooling of the adhesive before light-curing. Remove excess adhesive with a dry applicator brush or absorbent paper point. Following light curing of the adhesive, apply NX3 dual-cure resin cement onto the post, seal the post, vibrate the post slightly avoiding the possibility of trapped air then cure NX3 according to the manufacturer's instructions.

- Core Bonding** - Apply core material according to manufacturer's instructions for use. **Note:** Use a light-cured or dual-cured core material. Always light cure the first incremental build-up of core material.

\*Recommended Cure Times when using Kerr Demetron Lights:

Optilux 501 - 10 seconds  
L.E. Demetron I - 10 seconds  
L.E. Demetron II - 5 seconds  
Demi/Demi Plus/Demi Ultra - 5 seconds

For all other lights see manufacturer's recommendation.

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