

THE TIP TOP PROGRAM FOR ALL ULTRASONIC HANDPIECES FROM SIRONA

TIPS FOR EVERY PURPOSE, QUALITY IN EVERY DETAIL.



SIRONA.COM

The Dental Company

sirona.

THE GREAT EFFECT OF HIGH-QUALITY TIPS.

SMALL BUT SIGNIFICANT

Nearly all Sirona tips fit on all ultrasonic scalers from Sirona from any model range and any model year. This is always taken into account at the development stage.

THIS APPLIES TO:

- SIROSON S
- SIROSON L
- SIROSON C8
- SIROSONIC
- SIROSONIC L
- SIROSONIC TL
- PerioSonic

HIGH QUALITY

All Sirona tips are of the very highest quality. Only highly tempered stainless steel that has been conditioned and hardened in strict accordance with the Sirona tempering procedure is used in production. Sirona also continues to set new standards in quality by consistent use of state-of-the-art CNC processing technology.

| Technical data | PerioSonic/SIROSONIC/SIROSONIC L/ SIROSONIC TL |
|--|--|
| Drive | Piezoelectric |
| Oscillation | Linear |
| Oscillating frequency [kHz] | ~ 28–32 |
| Illumination* | LED |
| Ø of handpiece [mm] | max. 18 |
| Handpiece grip Ø [mm] | ~ 12 / ~ 18 |
| Handpiece length [mm] [without instrument tip] | ~ 96 / ~ 101 |
| Weight [g] | ~ 47 / ~ 50 |

* Not for SIROSONIC.

HIGH-QUALITY TIPS AND ULTRASONIC SCALERS.

A GREAT COMBINATION

The perfect result – regardless of the type of treatment – depends on the perfect combination of scaler and tip. By optimally matching the two components, Sirona ensures consistent high quality. This has produced ultrasonic scalers that are flexible and easy to use with a selection of tips designed to be used in specific treatment areas.

SIROSONIC/L/TL – HIGHLY VERSATILE: FLEXIBILITY INTEGRATED INTO THE DENTIST ELEMENT.

VERSATILE

The SIROSONIC/SIROSONIC L and SIROSONIC TL ultrasonic handpieces with powerful illumination are the perfect instruments for many types of treatment in the modern dental practice. Their reliable power regulator allows them to be used both for highly sensitive periodontal treatments and micropreparation. A Sirona spray adapter can also be attached to both handpieces for delivery of external rinsing fluid (e.g. NaOCl or CHX solution).



Sirona offers the appropriate tips for a wide range of applications:

- Scaling
- Periodontology
- Implantology
- Endodontics
- Retrograde root treatment
- Micropreparation
- Filling therapy

SAFE OPERATION


A unique feature of all Sirona tips is optimum oscillation excursion – customized to the specific requirements of individual treatment areas.

Scaling and periodontal tips oscillate, e.g. two-dimensionally or only in one plane. Lateral movements are avoided and the teeth are effectively protected during treatment. The cutting capacity is affected by the wear on the tip. This can easily and quickly be checked any time using a checkcard for tips.

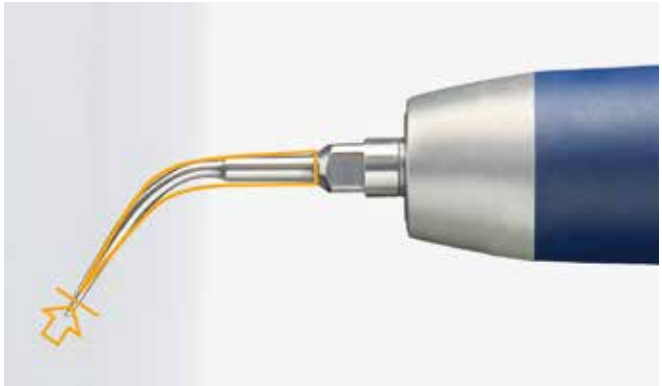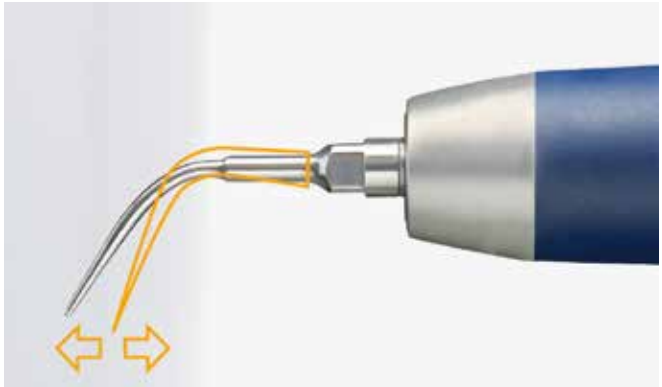

Up – down and forwards – backwards. The two-dimensional oscillation of the tip acts gently on the tooth being treated.




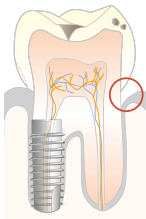

PERIODONTOLOGY

For scaling subgingival calculus, even in deep gingival pockets. The SIROPERIO tips provide optimum access in all areas for scaling calculus efficiently while being gentle on the tooth structure. Extremely convenient in use as it provides for good accessibility, even in bifurcations.

| Accessories | Figure | Usage | REF. |
|--|--------|--|-----------|
| SIROPERIO PE1* Straight, slender | | For rinsing and scaling roots in deep gingival pockets | 60 44 825 |
| SIROPERIO PE2* Curved to the right | | For scaling roots in deep gingival pockets in the posterior tooth region | 60 44 833 |
| SIROPERIO PE3* Curved to the left | | For scaling roots in deep gingival pockets in the posterior tooth region | 60 44 841 |
| SIROPERIO 4 PS* Straight | | For detection and scaling roots in deep gingival pockets with PerioScan | 61 77 005 |
| SIROPERIO PE5* Curved to the right | | For scaling roots in deep gingival pockets in the posterior tooth region | 63 43 078 |
| SIROPERIO PE6* Curved to the left | | For scaling roots in deep gingival pockets in the posterior tooth region | 63 43 086 |
| SIROPERIO PE7* Straight, slender with depth scale | | For rinsing and scaling roots in deep gingival pockets | 63 43 029 |
| SIROPERIO PE8* Straight, with depth scale | | For detection and scaling roots in deep gingival pockets with PerioScan | 63 43 045 |

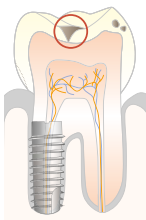


FILLING THERAPY

For gentle, safe insertion of inlays.

| Accessories | Figure | Usage | REF. |
|--|--------|---|-----------|
| CEM set no. 6L* (1 CEM tip no. 6L, 2 contact caps for inlays) | | for inserting ceramic inlays (e.g. CEREC) with thixotropic composites | 18 94 307 |
| Contact cap | | for CEM tip no. 6L, 10 pcs for inlays | 18 94 372 |


* Incl. torque wrench.

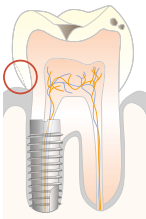

SCALING

Wide range of tips available for all stages of supragingival scaling.


| Accessories | Figure | Usage | REF. |
|---------------------|--------|---|-----------|
| Tip no. 1L* | | For initial lingual and buccal scaling | 89 24 250 |
| Tip no. 2L* | | Versatile, lingual, buccal and approximal | 89 24 268 |
| Tip no. 3L* | | Universal bilateral, subgingival and interdental | 18 94 455 |
| Set no. 3L* (2 pcs) | | see tip no. 3L* | 64 12 451 |
| Set no. 3L* (6 pcs) | | see tip no. 3L* | 65 41 044 |
| Tip no. 4L* | | Universal, ideal for supragingival and subgingival tooth surfaces and interdental areas | 18 94 463 |
| Set no. 4L* (2 pcs) | | see tip no. 4L* | 64 12 469 |
| Set no. 4L* (6 pcs) | | see tip no. 4L* | 65 41 051 |

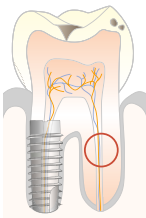

ENDODONTICS

For irrigating root canals which have had an application of sodium hypochlorite. Ultrasonic oscillations increase the effectiveness of the rinsing liquid and ensure its penetration even into capillaries.


| Accessories | Figure | Usage | REF. |
|---|--------|---------------------------------------|-----------|
| Endo tip no. 5 only for SIROSON L, SIROSONIC L/TL and PerioSonic | | for irrigating root canals with NaOCl | 60 02 310 |
| Endodontic set L | | only for SIROSON L, SIROSONIC L | 89 28 400 |

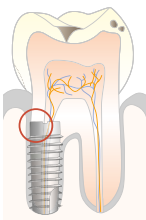

IMPLANTOLOGY

For optimal biofilm management and gentle cleaning of implants as well as metallic and ceramic restorations. Due to a special plastic coating, the cleaning insert operates smoothly on sensitive surfaces ensuring an optimal and gentle treatment.


| Accessories | Figure | Usage | REF. |
|--------------------------------|--------|---|-----------|
| SIROIMPLANT IP1 | | For cleaning of implants as well as metallic and ceramic restorations | 63 43 052 |
| Insert IP1 for SIROIMPLANT IP1 | | Scope of supply: 5 pcs | 63 43 094 |

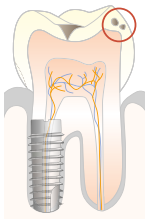

MICROPREPARATION

Selectively diamond-coated tips with a 20° bevel for minimally invasive preparation and adhesive treatment of small approximal cavities. Cooling spray is delivered directly to the preparation site. Contra-angle ensures universal access to all quadrants with only two tips of each size.

| Accessories | Figure | Usage | REF. |
|--------------------------|--------|--------|-----------|
| SIROPREP M1* Small | | Mesial | 59 04 276 |
| SIROPREP D1* Small | | Distal | 59 47 309 |
| SIROPREP M2* Standard | | Mesial | 59 47 358 |
| SIROPREP D2* Standard | | Distal | 59 47 366 |

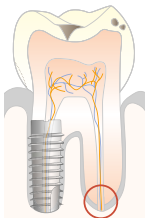

RETROGRADE ROOT TREATMENT

Selective, diamond-coated tips for retrograde apicoectomies. The tips are angled at approx. 20° (right or left) to facilitate access to the preparation site.

| Accessories | Figure | Usage | REF. |
|---|--------|--|-----------|
| SIRORETRO KR* Short, angled to the right | | For the lower left and upper right quadrants | 54 57 895 |
| SIRORETRO KL* Short, angled to the left | | For the lower right and upper left quadrants | 54 57 887 |
| SIRORETRO GR* Large, angled to the right | | For the lower left and upper right quadrants | 54 57 911 |
| SIRORETRO GL* Large, angled to the left | | For the lower right and upper left quadrants | 54 57 903 |

ALWAYS AT THE FOREFRONT OF INNOVATION!

As global innovation leader for dental equipment, we continuously invest in research and thus in the future of modern dentistry. By networking digital technologies with integrated solutions and optimizing the treatment workflow, we can create improved treatment results, more comfort and safety for the patient as well as time and cost savings in everyday work. The combination of constant innovative power and globally growing sales and service structures makes Sirona the global market leader trusted by thousands of practices and labs around the world. **Enjoy every day. With Sirona.**



CAD/CAM systems

From pioneer to new standard. For 30 years we have been developing digital dentistry and creating new possibilities for the future practice and lab.



Imaging systems

Best image quality with the lowest dose. More than 100 years of developing x-rays for the dental practice make us the number 1 innovation partner.



Treatment centers

The business card of modern practices. We are striving to create the ideal ergonomic and innovative center. Individually tailored to the well-being and demands of the patient and dentist.



Instruments

Advantages that speak for themselves. We make sure that we provide the right balance of proven quality, individual ergonomics and innovative technology for user-friendly work.



Hygiene systems

Competence that gives you safety. When it comes to hygiene in the practice, we do not take any shortcuts.