GREEN&CLEAN CL N

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EN – Instructions for use

Translations

Translation of the original operation instruction

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

Images

The images contained in these instructions for use are for reference and may differ from the actual appearance of the product.

1. Notes

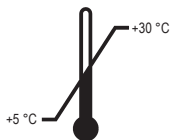
1.1. General instructions

Storage temperature

Product identifiers

Alkylethercarbonic acid (53563-70-5)
 Protease (9014-01-1)
 Lipase (9001-62-1)

1.2. Explanation of the symbols

1	CE mark	14	Observe instructions for use
2	Medical device	15	Use hand protection
3	REF/order number	16	Use eye protection
4	Batch designation	17	Wear protective clothing
5	Expiry date	18	Information
6	Manufacturer	19	Download
7	Follow instructions for use	20	Protect from heat / Protect from sunlight
8	Temperature limit	21	Protect from moisture / Store in a dry place
9	Quantity information	22	Fragile
10	Responsible for the translation	23	Package orientation at the top
11	Health hazard	24	Limited quantity
12	Corrosive	25	Stacking limit
13	Harmful to health		

1.3. Copyright notice

All names and contents are protected by copyright. Distribution, duplication or alternative use of this document is only permitted with the written consent of METASYS Medizintechnik GmbH.

2. Intended use

GREEN&CLEAN CL N is a concentrate for cleaning and deodorising the suction lines of dental suction systems.

Indication: Removal of blood and saliva in the waste water path of the dental suction system. To be used at the end of the working day and after long surgical procedures.

Contraindication: Not applicable.

Target group: Patients and trained medical personnel, without restriction as to gender and age, who come into contact with suction lines of dental suction systems.

3. Safety-related information

3.1. General safety-related information

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or the patient is resident.

3.2. Danger and safety information

The mixture is classified as hazardous according to (EC) regulation no. 1272/2008 (CLP).

Warning

H318	Causes serious eye damage.
H334	May cause allergy or asthma symptoms or breathing difficulties if inhaled.
H315	Causes skin irritation.
P101	If medical advice is needed, have product container or label at hand.
P102	Keep out of reach of children.
P261	Avoid breathing dust/fume/gas/mist/vapours/spray.
P280-metasy	Wear protective gloves/protective clothing/eye protection/face protection.
P304 + P340	IF INHALED: Remove person to fresh air and keep comfortable for breathing.
P342 + P311	If experiencing respiratory symptoms: Call a POISON CENTER/doctor.
P305 + P351 +	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if
P338	present and easy to do. Continue rinsing.
P310	Immediately call a POISON CENTER/doctor/...
P501	Dispose of contents/container to a suitable recycling or disposal facility.

3.3. First aid measures

Emergency number: Munich Poison Control +49 (0)89 19240

Description of the first aid measures

General instructions	Immediately remove contaminated, saturated clothing.
After inhalation	In the event of breathing difficulties, give oxygen. In the event of irregular breathing or respiratory arrest, initiate artificial respiration. Ensure fresh air. After inhalation of spray, seek medical advice immediately and show the packaging or label.
After skin contact	In case of skin irritation, consult a doctor. After contact with skin, wash immediately with plenty of soap and water.
After eye contact	In case of contact with eyes, rinse immediately with running water for 10 to 15 minutes with the eyelids open. Then consult an ophthalmologist.
After ingestion	Rinse mouth immediately and drink plenty of water. Do not induce vomiting. If swallowed, rinse the mouth with plenty of water (only if the person is conscious) and get medical help immediately.

3.4. Personal protective equipment (PPE)

Eye/Face protection	Suitable eye protection: Tightly sealing safety goggles Recommended eye protection brands: DIN EN 165
----------------------------	--

Hand protection	Suitable material: NBR (nitrile rubber). Thickness of the glove material: 0.4 mm Recommended glove brands: EN ISO 374 Additional hand protection measures: Wear cotton undergloves if possible.
Physical protection	Suitable physical protection: Laboratory coat
Respiratory protection	Respiratory protection is not required when used properly and under normal conditions.

4. Product description

4.1. Product description

GREEN&CLEAN CL N is a concentrate based on alkylethercarbonic acid, protease and lipase for daily cleaning of extraction systems.

4.2. Composition

Alkylethercarbonic acid combined with protease and lipase

4.3. Exposure time

The user solution is sucked in by the suction flow of the suction unit and should remain in the suction system for 60 minutes if possible. Longer exposure times are also possible (e.g. over the weekend).

5. Use

5.1. Dosage

Water	Solution
0,60 l	1.6 %

5.2. Application

Note

- > Rinse the suction unit with cold water after each treatment
- > Wear personal protective equipment during application
- > Adhere to the dosage
- > Mixed solution is intended for immediate use

Application

Open the sealing cap of the dispensing container attached to the top of the bottle.

26

Fill the container with 10 ml of the concentrate by gently pressing the bottle.

Only dispense when the sealing cap is open.

Empty the 10 ml GREEN&CLEAN CL N into the mixing container.

27 Fill the container up to the mark with cold water (= 600 ml 1.6% solution)

28 Hold the small suction hose on opening no. 1 until air is sucked in (200 ml)

29 Hold the large suction hose on opening no. 2 until air is sucked in (200 ml)

30 Empty the remaining solution into the rinsing basin (200 ml). Do not rinse.

31 Wait for reaction time (60 min) and then rinse with cold water

Allow GREEN&CLEAN CL N to work overnight and over the weekend. If necessary (e.g. in case of high soiling), we recommend an additional application at noon.

6. Disposal

Dispose of product in compliance with official regulations.

	before intended use	after intended use
	Product	Packaging
Product waste code	70699	150102
Hazardous waste	No	No
Waste designation	Waste not otherwise specified	Plastic packaging

7. Annex

7.1. REF numbers and scope of delivery

REF	Designation
60010305	GREEN&CLEAN CL N, 2 x 1000 ml bottles
40200023	Mixing container for GREEN&CLEAN CL N

7.2. Change history

Revision	Date	Description
ZK-58.102/00	22.06.2021	Newly created.

Subject to printing and setting errors!

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Traductions

Traduction de la notice originale

Tetras GmbH

Sirius Business Park, Rupert-Mayer-Str. 44, 81379 Munich, Germany

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