



Instruction BPR Vacu-Bag Ports / Carts


The disposable bag „BPR Vacu-Bag Ports“ is used for the ports and the „BPR Vacu-Bag Carts“ for the carts.

DESCRIPTION	PICTURES
<ul style="list-style-type: none">The BPR Vacu-Bag Ports / Carts and the vacuum tank are placed ready.	
<ul style="list-style-type: none">The BPR Vacu-Bag Ports / Carts is inserted into the vacuum tank and aligned.	

- The bottleneck is screwed onto the vacuum tank.



- After treatment, the bottleneck is unscrewed.
- The BPR Vacu-Bag Ports / Carts is then carefully removed from the vacuum tank.

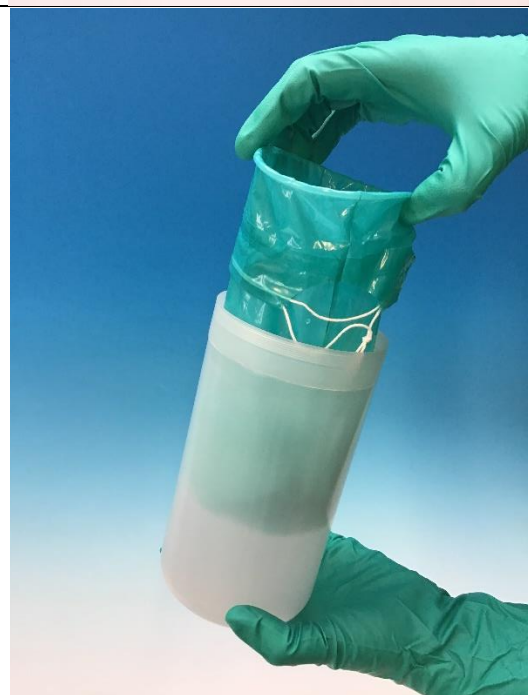

- The BPR Vacu-Bag Ports / Carts is closed with the drawstring provided.



- The BPR Vacu-Bag Ports / Carts can now be disposed of in a leak proof manner.

