Polishing Systems

Top Dent Composite Pre-Polishers

Top Dent Universal Pre-Polishers

Top Dent Gold/Amalgam/Metal Pre-Polishers

Top Dent Composite High-Gloss Polishers

GENERAL INFORMATION

Top Dent Polishers are designed for finishing and polishing of restorative materials in order to achieve an aesthetic dental restoration. Various shapes of polishers are ideal for use on all surfaces – convex, concave, occlusal, and proximal. Polishers are single use devices.

PRODUCT COMPOSITION

Polishers are composed of: silicone rubber, silicone carbide, aluminum oxide, diamond particles, brass mandrel with nickel or gold plating. The shank geometry is Type 1 according to ISO 1797.

INDICATIONS FOR USE

Top Dent Polishers are designed for prepolishing, finishing and high-gloss polishing of direct and indirect human dental restorations to rebuild the healthy anatomical shape of the tooth.

CONTRAINDICATIONS

Top Dent Polishers should not be used on patients who have a history of severe allergic reaction to any of their components or materials (see "Product Composition"). If skin rash, irritation, sensitization or allergic reactions occur, seek immediate medical attention.

WARNINGS

- Top Dent Polishers are for single use only. Do not reuse this device.
- This device is sold non-sterile and needs to be sterilized prior to use by following instructions in the "Sterilization" section.
- When using this device, observe adequate Revolutions Per Minute (RPM) and the correct direction of the revolutions to prevent potential device malfunction and tooth overheating due to excessive RPMs.

PRECAUTIONS

- This product is intended to be used by qualified dental professionals on the general patient population.
- Use of personal protective equipment is recommended when using this device.
- As much as possible to prevent soft tissue injury, do not use this device in contact with soft tissues.
- Handle the device carefully during use. Dropping, pulling or over twisting could damage the device.
- Always use a spray coolant to prevent tooth overheating.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS

- Connect the polisher to a latch-type dental contra-angle.
- Set the RPM at clockwise rotation – a maximum of 10'000 RPMs.
- Use the polisher with spray cooling to polish dental restorations.
- Discard the device after single use.

STERILIZATION

Instructions for Sterilizing Top Dent Polishers																					
WARNINGS	<ul style="list-style-type: none">This device is sold non-sterile and needs to be sterilized prior to first use. During sterilization, avoid direct contact with the metallic part of the autoclave as the temperature there can exceed the pre-set temperature.																				
Limitations on Processing	Do not use chemical autoclave or cold sterilization solutions.																				
Initial treatment at the point of use	Remove the polisher from the package.																				
Packaging	Paper / plastic steam sterilization pouches may be used. Use a pouch that is FDA-cleared or ISO 11607 compliant.																				
Sterilization	Place the component in a sterilization pouch suitable for steam sterilization following the instructions of the pouch manufacturer. Pay attention to the tip of the instruments as it may damage the packaging during handling, distribution or storage. Proper packaging protects the sterilized instruments from bacterial recontamination during handling and storage. <p>Use a steam autoclave with distilled water. Do not exceed 135°C (275°F).</p> <p>Autoclave Parameters</p> <table> <tbody><tr> <th>Region</th> <th>Autoclave Type</th> <th>Cycle Temperature</th> <th>Exposure Time</th> <th>Minimum Dry Time</th></tr> <tr> <td>EU</td> <td>Pre-vacuum</td> <td>134°C (273°F)</td> <td>3 minutes</td> <td>14 minutes</td></tr> <tr> <td>US</td> <td>Gravity (Wrapped)</td> <td>121°C (250°F)</td> <td>30 minutes</td> <td>15-30 minutes</td></tr> <tr> <td>US</td> <td>Pre-Vacuum (Wrapped)</td> <td>135°C (275°F)</td> <td>3 minutes</td> <td>16 minutes</td></tr> </tbody></table> <p>Note: Ensure that your sterilizer is subjected to a preventive maintenance plan according to the indications given by the manufacturer of the steam sterilizer. The use of a steam sterilizer complying with EN 285 or EN 13060 is highly recommended. Health care facilities are responsible for the training of staff on infection control and sterilization according to the manufacturers' instructions.</p>	Region	Autoclave Type	Cycle Temperature	Exposure Time	Minimum Dry Time	EU	Pre-vacuum	134°C (273°F)	3 minutes	14 minutes	US	Gravity (Wrapped)	121°C (250°F)	30 minutes	15-30 minutes	US	Pre-Vacuum (Wrapped)	135°C (275°F)	3 minutes	16 minutes
Region	Autoclave Type	Cycle Temperature	Exposure Time	Minimum Dry Time																	
EU	Pre-vacuum	134°C (273°F)	3 minutes	14 minutes																	
US	Gravity (Wrapped)	121°C (250°F)	30 minutes	15-30 minutes																	
US	Pre-Vacuum (Wrapped)	135°C (275°F)	3 minutes	16 minutes																	
Storage	When the product is taken out of the autoclave and before use, check the package for damage. In case of perforated package, re-pack the product and repeat the sterilization procedure. Store according to "Storage and Disposal".																				
Manufacturer Contact	See Manufacturer contact details on the IFU.																				

STORAGE AND DISPOSAL

Keep away from sunlight. Refer to outer package for expiration date. Do not use after expiration date. For proper disposal always follow local and national regulations.

TROUBLESHOOTING AND TECHNICAL SUPPORT (IF REQUIRED)

Not applicable.

DISCLAIMER

The instructions provided above have been approved by Kerr Corporation as being CAPABLE of preparing a medical device, including reusable ones. It remains the responsibility of the dental professional / user to ensure that the reprocessing actually performed by equipment, materials and personnel is done per the instructions provided above in order to achieve the desired result. Any deviation by the dental professional / user from the instructions provided above will be done at your own risk; the Kerr Corporation will not be able to address any requests for refund or exchanges under guarantee of any products that have not been handled or reprocessed in accordance with the above instructions.

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TOP DENT

Polishing Systems

Composite Pre-Polishers | Universal Pre-Polishers

Gold/Amalgam/Metal Pre-Polishers

Composite High-Gloss Polishers

INSTRUCTIONS FOR USE

Ormo Co B.V.
Basicweg 20 NL-3821 BR
Amersfoort, The Netherlands

Kerr Corporation **Kerr Italia S.r.l.**
1717 West Collins Ave. Via Passanti, 174Scafati (SA)
Orange, CA 92867 USA 84018, Italy

SYMBOLS DESCRIPTION

Full explanation of symbols used on the packaging located at: http://www.kerrdental.com/symbols-glossary

	Catalogue number
	Manufacturer
	Use -by date
	European conformity (CE) mark with Notified Body identification number for Class Im, Ir, Is, Ila, I Ib, III medical devices. Notified Body No. 2797: BSI, The Netherlands
	Consult Instructions for Use
	Caution
	European Authorized Representative
	Medical Device
	Spray-cooling
	Wear eye protection
	Right Revolution (clockwise rotation): maximum RPM indicated on the label
	Sterilizable in a steam sterilizer (autoclave) at temperature specified
	Keep away from sunlight
	Importer

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