EN

Instructions

Preparation before cleaning

Wipe off the instrument after use to prevent soil and debris to dry onto the instrument.
Perform cleaning as soon as possible after use. Do not exceed 2 hours.

Cleaning Manual

Equipment
Soft bristle brushes of various sizes, Ultrasonic bath
Detergent
Follow agent manufacturer's recommendations for concentration and temperature.
Manual: Enzymatic or low-alkaline (pH<8) detergent suitable for manual cleaning.
Ultrasonic bath: Enzymatic or low-alkaline detergent with minimal foaming characteristics.
Water quality
Drinking water quality for manual cleaning and distilled or de-mineralized water for ultrasonic bath and final rinse should be used.
Instruction
1. Immerse in freshly prepared cleaning solution as per manufacturer's instructions allow maximum 40°C. Clean mechanically with a brush, working beneath the liquid level, until visibly clean.
2. Rinse thoroughly
3. Clean in an ultrasonic bath for a minimum of 5 minutes, using a frequency of 35-45 KHz and power min. 150 W. Make sure the instruments are in an open position.
4. Rinse thoroughly for min 30 seconds.
4a Carefully dry instruments with clean compressed air (Class 1 or better according ISO 8573-1:2010) or lint free towel. Any remaining water can cause potential rust or corrosion on the instrument surface and into the substrate.
5. Inspect cleanliness. If necessary, repeat from step 1.

Disinfection Manual

Detergent
Immerse into suitable disinfection solution. Follow the manufacturer's instruction of disinfection solution for concentration and time.

Detergent
Non-corrosive, neutral cleaning agent with minimal foaming characteristics, suited for medical devices.
Follow agent manufacturer's instructions regarding concentration and temperature.

Water quality
Drinking water quality for cleaning and distilled or purified water for final rinse/disinfection.

Instructions
1. Load the instruments in an open position in the washer-disinfector. Use a suitable instrument tray.
2. Run program:
- Pre-wash in cold water, 2 x 2 min
- Main wash with detergent at minimum 55°C, 10 min
- Rinse in warm water, 2 x 1 min.
- Final rinse/disinfection in distilled or de-mineralized water at 90°C, minimum 1 min
- Drying at 110°C, minimum 15 min
When necessary, control that the instruments are clean. If required, repeat from step 1 or use manual cleaning.

Inspection and maintenance

All plastic products have a limited-service lifetime and should be replaced at regular intervals. All types of cleaning reduce the product's service lifetime. The product should be replaced when its function and/or performance no longer satisfies the specifications in this manual.

Packaging for sterilization

Equipment
Instruction
Standard packaging pouch
Pack the pouches in an open position in the pouch. Place instruments plastic towards plastic and paper towards paper.

Sterilization
Caution
Equipment
Instructions
The instrument must be cleaned and disinfected before sterilization.

Steam autoclave (validated according to EN 13060, EN 285, EN ISO 17665-1)
Run minimal cycle
Steam temperature Min. 134 °C / 273 °F
Steam exposure time 4 min
Vacuum drying 3 min

Storage

Ensure the instruments are dry before storage. The instrument should be stored in a packed tray, free from dust and moisture and under moderate temperatures of 5°C - 40°C.

Transportation

If required, secure transport in closed boxes is recommended in order to avoid any damage to the medical device as well as environmental contamination

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