



INSTRUCTION FOR PRODUCTS MADE OF ALUMINIUM

- DON'T PUT INTO WASHING-MACHINE
- DON'T USE ULTRASOUND
- DON'T USE THERMODISINFECTOR
- ATTENTION: some products of decontamination are NOT COMPATIBLE with aluminium: follow always the enclosed instructions.

The articles made of aluminium are coloured by a process of anodizing. The anodizing and the aluminium can be damaged by some agents.

Don't use alkaline products, some products too acid, or product with Sodium or Potassium.

The PH from 4 to 8 doesn't produce any particular risk.

Follow the enclosed instruction of the products you are going to use for the cleaning and remember the incompatibility with the aluminium.

The use of ultrasound basin for the cleaning risks to deteriorate the anodizing and the colour too.

The products which will presents signs of deterioration connected with a bad manipulation CANNOT be changed.

The following way of sterilization can be used in safeness, always following the enclosed instruction of the producer:

- Autoclave – 134°
- Chemiclave – 132°
- Dry heat – 180°